

MethChek 100 Quick Guide

Immunoassay Wipe Test Kit (100 ng LLOI)

This Quick Guide provides only a brief overview of the procedure. Read complete operating instructions for details prior to first use of MethChek® 100.

Wear supplied gloves for all procedures.



Place

items on wax paper.



Place

template on test surface.



Spray

Step 1 Wetting Agent 2 times.




Wipe

test surface.



Extract

Insert gauze in syringe, pour Step 2 Extractor in syringe, express liquid into tray, draw liquid back, express again.

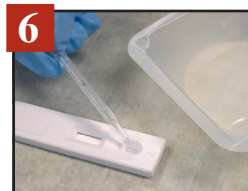

Pipette

3 drops tray liquid into "S" well on Step 3 Cartridge.

7 Read Results

Position cartridge with sample well on right and observe. Results appear in 1 min. and complete in 5 mins.




Positive, meth residue detected above 100 ng LLOI



Negative, meth residue not detected at 100 ng LLOI
(See Operating Instructions for details.)



Invalid Test, perform new test

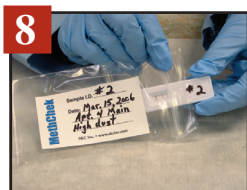

Identify

Mark ID on cartridge and mini bag. Insert cartridge in bag for transport or storage.



Dispose

Place disposable items in resealable bag. Dispose observing federal, state, and local environmental regulations.

